## DV51 Standard CPAP Series

### EN DeVilbiss® SleepCube™ Positive Airway Pressure Device

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
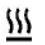
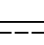
## TABLE OF CONTENTS

Symbol Definitions .....	3
Important Safeguards .....	3
Introduction.....	4
Intended Use/Indications for Use.....	4
Contraindications.....	4
What Is Obstructive Sleep Apnea (OSA).....	4
How CPAP Therapy Works.....	5
Acclimating To Your Therapy .....	5
Benefits Of Your Therapy .....	5
Helpful Resources .....	6
Key Features.....	6
Unpacking The Contents .....	6
SleepCube Device.....	6
Keypad.....	7
System Assembly without Humidification.....	7
Keypad and Display .....	7
Operation .....	8
Start Up.....	8
Using The Comfort Delay Feature .....	8
Shut Down .....	8
Advanced Menu Controls .....	8
Advanced Menu List.....	9
Enable Menu List.....	9
CPAP Patient Menu.....	10
Patient Messages .....	10
Reminders .....	10
Notifications .....	11
SleepCube Travel Information.....	11
SleepCube DC Operation.....	11
Battery Power .....	11
Supplemental Oxygen .....	11
Accessories/Replacement Items .....	12
Maintenance .....	13
Cleaning.....	13
Troubleshooting.....	14
Specifications .....	15
Declaration of Conformity.....	16

## SYMBOL DEFINITIONS

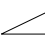 Class II electrical protection—double insulated	 Type BF equipment— applied part
 Data Port Input/Output	 Standby – turns the blower on or off
 Next Item on LCD display	 Increase value shown on LCD display
 Previous Item on LCD display	 Decrease value shown on LCD display
 Attention - Consult Accompanying Documents	 Heat
 DC Input: 12 VDC, center pin is positive	 100 – 240 Volts AC input 50/60 Hz
 Locked	 Unlocked
 IPX1 Drip-Proof, Vertical	 Delay – starts the delay cycle, if prescribed
 This device contains electrical and/or electronic equipment that must be recycled per EC Directive 2002/96/EC - Waste Electrical and Electronic Equipment (WEEE)	

## IMPORTANT SAFEGUARDS

When using electrical products, basic safety precautions should always be followed. Read all instructions before using this device. Important information is highlighted by the following terms.

**DANGER** Urgent safety information for hazards that will cause serious injury or death.

**WARNING** Important safety information for hazards that might cause serious injury.

**CAUTION** Information for preventing damage to the product.

**NOTE** Information to which you should pay special attention.

**PLEASE READ ALL INSTRUCTIONS BEFORE USING THIS DEVICE.**

**SAVE THESE INSTRUCTIONS**

### DANGER

**Electric Shock Hazard – Do not use while bathing.**

**Electric Shock Hazard – Do not immerse this device into water or any other liquid.**

**Electric Shock Hazard – Do not attempt to open or remove the enclosure. There are no user-serviceable internal components. If service is required, return the product to your home care provider. Opening or tampering with the product will void the warranty.**

### WARNING

**The DeVilbiss SleepCube™ should be used only with masks recommended by DeVilbiss, your physician or respiratory therapist.**

**To avoid rebreathing of exhaled air, do not use a CPAP mask unless the device is turned on and providing a supply of air. Venting in the mask should never be blocked. When the device is turned on and providing a fresh supply of air, exhaled air is flushed out of the mask vent. However, when the device is not operating, exhaled air may be rebreathed. Rebreathing of exhaled air for longer than several minutes can in some circumstances lead to suffocation. This warning applies to most CPAP devices.**

**The DeVilbiss SleepCube is not a life support device and may stop operating with certain device faults or with a power**

**failure. It is intended to be used on spontaneously breathing individuals weighing 66 lbs/30 Kg or greater.**

**To avoid electric shock, always unplug power cord from wall outlet power source when performing cleaning.**

**Use only accessories recommended by DeVilbiss.**

**CAUTION**—*The circular data port connector located on the back of the SleepCube is used to attach accessories to the device. The connector must only be used with accessories approved for use by DeVilbiss. Do not attempt to attach any other device to this connector as it may damage the CPAP or the accessory device.*

**CAUTION**—*Never rinse or place the device in water. Never allow liquids to get into or around any of the ports, switches or air filter; doing so will result in device damage. If this occurs, discontinue use and remove the power cord from the power source. Allow the device to completely dry before use.*

**CAUTION**—*Do not place the SleepCube where it can be bumped onto the floor or where the power cord may create a trip hazard.*

**CAUTION**—*Only the DeVilbiss DV5 series Heated Humidifier system is recommended for use with the SleepCube. Other humidifier systems may prevent the device from detecting snoring and may cause inappropriate pressure levels in the mask.*

**CAUTION**—*Oxygen is a prescription gas and should only be administered under the supervision of a physician.*

## INTRODUCTION

### Intended Use/Indications for Use

The DeVilbiss® SleepCube™ Model DV5 Series is intended for use in treating obstructive sleep apnea in patients 66 lbs/30 kg and above.

### Contraindication

Do not use SleepCube if your upper airway has been bypassed.


**NOTE**—*Symptoms of dryness of the throat, nasal passages, and the mouth are common with positive airway pressure therapy. This is especially true in dry climates and during the cold season when humidity in the air is typically lower than at other times. The SleepCube features an optional humidifier system to help minimize these effects.*

### What Is Obstructive Sleep Apnea (OSA)?

Obstructive Sleep Apnea (OSA) is a breathing disorder that affects more than 22 million Americans – approximately 4% of middle-aged men and 2% of middle-aged women.

Physiological factors such as size of tonsils, shape of the nose, narrowing of the upper airway, or any combination of these can contribute to the potential for having OSA. During sleep, your airway can relax so much that it begins to collapse and becomes obstructed. Relaxation allows the tissue at the back of your throat to vibrate as you breathe causing snoring. But as your airway collapses more fully, it blocks your airflow and stops your breathing altogether. (In fact, the word “apnea” is derived from a Greek term meaning “without breath.”)

This stoppage occurs repeatedly throughout the night, each episode lasting from just a few seconds to more than a minute. As your blood oxygen level drops to dangerous levels, your central nervous system triggers a sudden gasp for breath, partially awakening you and preventing you from experiencing the sustained, deep sleep your body needs. As a result, it is common to feel fatigued, stressed and irritable the following day – a less-than-optimal and potentially dangerous condition in which to work, drive, and make decisions.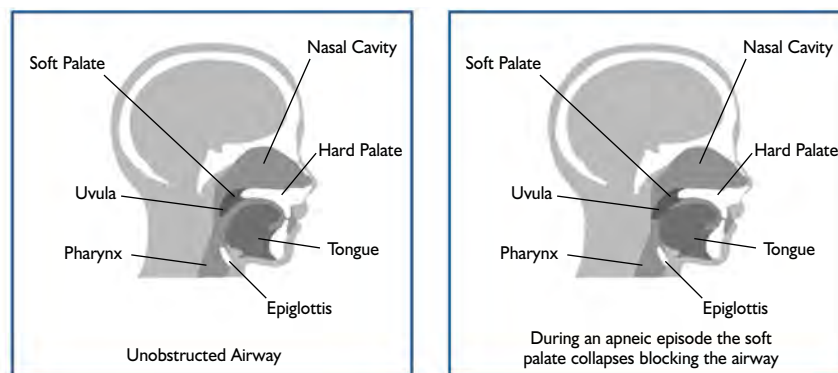

There are important medical concerns as well. OSA starves your heart, brain, and organs of life-sustaining oxygen. Over time, this deprivation can seriously impact your health potentially resulting in memory loss, impotence, hypertension, coronary disease, strokes, and heart attacks. Recent studies indicate that OSA has the same risk factor for heart disease as smoking, high cholesterol and alcohol, and estimate that as many as 38,000 people a year die from the effects of OSA.

Clearly, OSA is a serious medical condition that requires attention and treatment.

## How CPAP Therapy Works

There are several treatments for OSA including surgery and CPAP therapy. However, CPAP therapy has proven very effective for most patients. CPAP stands for “continuous positive airway pressure” which is exactly what the DeVilbiss SleepCube device delivers to your upper airway, acting as an “air splint” to keep the passage open while you sleep.

This reduces or eliminates the obstruction, allowing you to enjoy the deep, refreshing, uninterrupted sleep you need. More importantly, your body will get the rest and oxygen it needs with more continuous, uninterrupted sleep – helping you avoid some of the serious health risks associated with OSA. And the SleepCube is designed to provide this therapy to you quietly and comfortably so that you can sleep through the night without distraction.

## Acclimating To Your Therapy

Naturally anything new – including CPAP therapy – takes some getting used to. Having your mouth or nose (or both) covered by a mask may make you feel anxious or claustrophobic. Feeling the rush of air from your SleepCube may cause discomfort or concern. But don't worry – every successful, long-term CPAP user began where you are today with the same reactions and concerns, and overcame them to enjoy the full benefits of their therapy.

Here are some steps to help you gradually ease into your therapy and overcome your anxieties:

1. Try attaching the mask to the CPAP unit, and switching the unit “On.” Practice breathing through the mask for one hour while watching television, reading, or performing some other sitting activity.
2. If you nap, use the CPAP during this time.
3. Use CPAP at least during the first initial 3-4 hours of nighttime sleep.
4. Use CPAP through an entire night of sleep.

Advance through the above steps one step at a time and progress to the next step as soon as you can carry out the current step without anxiety.

## Benefits Of Your Therapy

As a DeVilbiss SleepCube user, you are on your way to living a happier, healthier life – a life free from the sleep disruptions and health risks associated with OSA. After you become acclimated to CPAP therapy, you may be surprised at how much better you feel, both physically and emotionally. CPAP therapy may give you the opportunity to finally experience the deep, restful sleep you've been missing out on. Many patients find they have more energy, better judgment, and improved concentration during the day. With effective therapy you'll be less stressed, less irritable and more like your old self again, with a healthy, positive outlook on life. In fact, you may well wish you'd gotten your SleepCube sooner!

## Helpful Resources

As you begin your SleepCube therapy, remember: you're not in this alone! You're part of a large and growing network of CPAP users, and many resources are available to answer your questions, address your concerns, and help make your therapy as effective and effortless as possible.

### American Academy of Sleep Medicine

One Westbrook Corporate Ctr, Ste 920  
Westchester IL 60154  
Phone: (708) 492-0390  
Fax: (708) 492-0943  
www.aasmnet.org

### American College of Chest Physicians

3300 Dundee Rd  
Northbrook, IL 60062-2348  
Phone: (847) 498-1400 or (800) 343-2227  
Fax: (847) 498-5460  
www.chestnet.org

### The Sleep Apnoea Trust

12a Bakers Piece,  
Kingston Blount  
Oxon.  
OX39 4SW  
Tel: +44 845 60 60 685  
www.sleepapnoea-trust.org

### National Heart, Lung & Blood Institute Information Center

P.O. Box 30105  
Bethesda, MD 20824-0105  
Phone: (301) 592-8573  
Fax: (240) 629-3246  
www.nhlbi.nih.gov

### American Academy of Neurology

1080 Montreal Avenue  
St. Paul, Minnesota 55116  
Phone: (651) 695-2717 or (800) 879-1960  
Fax: (651) 695-2791  
www.aan.com

### American Sleep Apnea Association

1424 K Street NW, Suite 302  
Washington, DC 20005  
Phone: (202) 293-3650  
Fax: (202) 293-3656  
www.sleepapnea.org

### National Center on Sleep Disorder Research National Heart Lung & Blood Institute (NIH)

6701 Rockledge Drive  
Bethesda, MD 20892  
Phone: (301) 435-0199  
Fax: (301) 480-3451  
www.nhlbi.nih.gov/sleep

### National Sleep Foundation

1522 K Street, NW, Suite 500  
Washington, DC 20005  
Phone: (202) 347-3471  
Fax: (202) 347-3472  
www.sleepfoundation.org

## KEY FEATURES

### Unpacking The Contents

The following items are included in your DeVilbiss SleepCube system:



1. SleepCube device
2. Spare filter pack
3. AC line cord
4. Carrying case
5. Air supply tubing
6. User manual
7. DVD

If any of the above items are missing, please contact your home care provider.

**NOTE**—If you received a humidifier with your SleepCube system, please refer to the humidifier's instruction guide for additional information.

### DeVilbiss SleepCube Device (Figures A and B)


1. Keypad/LCD Display (see Figure C)
2. Air Supply Port on Back
3. Air Supply Port on Bottom (for optional humidifier)
4. Air Supply Port Plug
5. Heater Power Connector (for optional humidifier)
6. AC Power Connector
7. DC Power Connector
8. Data Port
9. Power Cord (appropriate to your wall outlet) (not shown)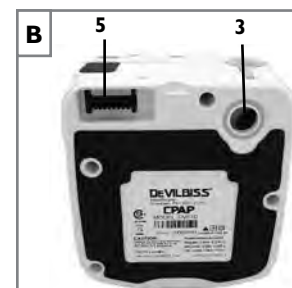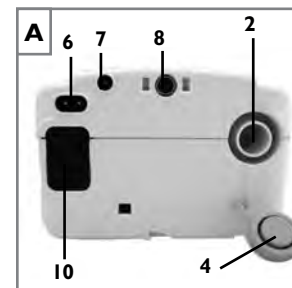

- 10. Air Inlet Filter Opening
- 11. Connector Cover (not shown)

## Keypad (Figure C)

- 1. On/Off
- 2. Previous Item
- 3. Next Item
- 4. Delay
- 5. Decrease Value
- 6. Increase Value
- 7. Heater Power LED (for optional humidifier)

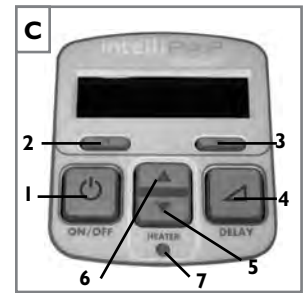Ask your home care provider for information about other DeVilbiss masks, equipment, and accessories.

## SYSTEM ASSEMBLY WITHOUT HUMIDIFICATION

**NOTE**—If your system includes a humidifier, please refer to the instructions provided with the humidifier for assembly and operation.

- 1. Locate the air supply port and heater accessory connector on the bottom of the SleepCube device. Ensure that each opening has the appropriate soft cover securely attached.
- 2. Place the SleepCube on a stable surface such as a nightstand or table. If you wish, you may also place the device on the floor at the bedside, being careful to place the unit where it won't be kicked or stepped on. Ensure that the air inlet in the back of the device is not blocked by anything such as curtains or bedding.

**NOTE**—Never place the SleepCube system on a soft surface such as a bed or couch during operation.

**CAUTION**—The height of the SleepCube system must always be lower than the interface mask when using a humidifier to prevent water from getting into the CPAP mask.

- 3. Insert the power cord into the back of the device. Make sure the power cord is fully inserted into the connector. Plug the other end of the power cord into the AC wall outlet.
- 4. Attach the air supply tubing to the air supply interface in the back of the device.
- 5. Prepare the mask (supplied by your home care provider) by referring to the mask's instruction guide.
- 6. Connect the air supply tubing to the mask.

**NOTE**—The air supply tubing supplied with the device is specially designed to prevent kinking and allow you to receive optimal airflow. Use only the 22 mm diameter, smooth-bore, kink-resistant tubing supplied with this device or an air supply tubing recommended by your home care provider.

## Keypad And Display

Basic DeVilbiss SleepCube Controls		
Name	Symbol	Function
ON/OFF		The ON/OFF button starts and stops the flow of air from the device.
HEATER		If your SleepCube has a heater/ humidifier attached, these buttons adjust the heater setting. Increase the setting for more humidity and decrease the setting for less humidity.
DELAY		The Delay button activates the comfort delay feature and only functions if this feature has been enabled. To understand how to set the parameters of this feature, please refer to the Using the Comfort Delay Feature found on page 8.

**NOTE**—The additional control buttons that you see on the SleepCube control panel provide advanced controls for specific settings. Please refer to the 'Advanced Menu Controls' section on page 8 for more details.

## OPERATION

### Startup

To begin operation, verify that your SleepCube is connected to a 'live' power source. When plugged in, the LCD display should appear as noted.

 OFF

Put on your CPAP mask and press the ON/OFF button on the keypad to turn on airflow OR put on your CPAP mask and take a few breaths to turn on airflow automatically. The pressure in your mask will rise to the set pressure in about 10 seconds.

**NOTE**—If the device does not automatically start after taking 2-3 breaths, this feature may have been disabled. If Auto ON/OFF is disabled, you must manually press the ON/OFF button to start and stop the device.

The keypad display shows the actual mask pressure while the device is in use. Because this is the actual pressure, the device pressure reading may vary slightly as you breathe. Your display should appear as noted.

 P:10.0 CPAP

If your DeVilbiss SleepCube system includes a humidifier, its heater controls and status are displayed as noted in the image.

 P:10.0 CPAP  
Heat:Off


**NOTE**—If the display on your device is not similar to those listed above, refer to the chapter section titled "Patient Messages" on page 11.

### Using The Comfort Delay Feature

Your SleepCube has a comfort delay feature to help make it easier for you to fall asleep. The airflow will start at a low pressure and then gradually ramp-up to the prescribed setting over the last few minutes of the specified delay time.

**NOTE**—When the delay comfort feature is activated, a message will flash on the display periodically, indicating the amount of time remaining in the delay.

You can adjust the amount of time spent in comfort delay by using the advanced device controls. The Delay Time can be adjusted from 0 to 45 minutes in 5-minute increments. Refer to page 9 for details.

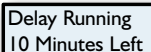 Delay Running  
10 Minutes Left

**NOTE**—If you are unable to change the delay time, your provider has locked this setting. Refer to the keypad menu guide on page 9 for details.

Press the delay button to activate the comfort delay feature. The airflow pressure will drop to the prescribed comfort delay pressure.

In the final 10 minutes of the delay period, the pressure will gently rise to the pressure prescribed for your therapy. You may repeat the comfort delay by pressing the delay button again if you have not fallen asleep during the first use.

### Shut Down

Press the ON/OFF button to turn the device off. You may also simply remove your mask. After a few seconds the device will display the message "Mask Off. Please check mask fit." If you do not put the mask on again the device will automatically turn off in about 20 seconds. Pressing any button will remove the message from the display.

**NOTE**—If the DeVilbiss SleepCube does not turn off automatically, the "Auto-OFF" feature may have been disabled. In this case you must manually turn the device off.

## ADVANCED MENU CONTROLS

With the device on, press the "Next" or "Previous" buttons on the keypad to scroll through the items on the display. The table shows the items you may see, and describes the feature associated with each item.

Name	Symbol	Function
PREVIOUS ITEM	◀	Pressing this button displays the previous menu option.
NEXT ITEM	▶	Pressing this button displays the next menu option.
SETTING CHANGE	⬆ ⬇ ⬆	These buttons allow you to adjust the setting that is shown on the display.

Please note that the values displayed are examples only.

## Advanced Menu List

Display Example	Description
P: 5.0 CPAP Heat:Off	This is the default screen when the SleepCube is turned on. The actual mask pressure and heater setting are shown. The displayed pressure may vary slightly while breathing. If the heater system is not connected, the heat setting will be blank.
Set Pressure 10.0 cmH2O	This is a prescription setting which controls the operating pressure.
Delay Time 45 minutes	Selectable delay time setting. Your provider has set the delay time. If the unlocked symbol is shown, you are able to adjust this setting. If the locked symbol is shown, this setting can only be adjusted by your provider.
Delay Pressure 5.0 cmH2O	This is a prescription setting which controls the operating pressure during the delay time.
Enable Menu Press ▼	Pressing the down key enters the Enable menu. Refer to next section for Enable Menu details.
LCD Contrast Adj 50 %	This setting changes the contrast for the LCD display. This only needs to be adjusted if you find it difficult to see the display from your viewing angle.
Compliance Meter 123456.7 hours	The total number of hours (in 1/10 hour increments) that the SleepCube has been providing therapy.
Hourmeter 123456.7 hours	The total number of hours (in 1/10 hour increments) that the SleepCube has been in operation.
Firmware Version V0.01 01/06/2006	Shows version of installed firmware.
Serial Number HD123456	Shows the serial number of the device.
Adherence Score 83% (25/30)	Shows the maximum adherence achieved for any period of thirty consecutive days of use within a ninety-day time frame. The score represents the percentage of days, within thirty consecutive days, where usage met or exceeded the 'Usage Threshold.'
SmartCode 1-Day G201-0Y7-6GPH	SmartCode™ is used for therapy management reporting; therapy information is encoded in a series of letters and numbers, and includes a special character for error checking to ensure accuracy. Use the UP arrow or DOWN arrow keys to change the SmartCode reporting period.

## Enable Menu List


The Enable menu is available in all modes and is a sub-menu that allows you to control various functions of your SleepCube. It is accessed by pressing the down button while in this Advanced Menu.

Display Example	Description
Auto-ON Enabled	This option controls the Auto-ON feature, which automatically starts the flow of air after breathing once or twice into the mask.
Auto-OFF Enabled	This option controls the Auto-OFF feature, which automatically stops the flow of air after taking off the mask. The device may require up to 30 seconds to turn OFF depending on the kind of mask used.
Exit Menu Press ▲	Pressing the up key exits the Enable menu.

## Patient Menu Layout

The following chart shows by Mode each Display you will scroll through using the Next or Previous buttons on the keypad. Pressing the Down key at the Enable Menu Display will allow you to enter this submenu - then use Next or Previous buttons to scroll through it; press the Up button to return to main menu. Once the final Display is reached, pressing the Next button will return you to the first Display.

### CPAP MENU

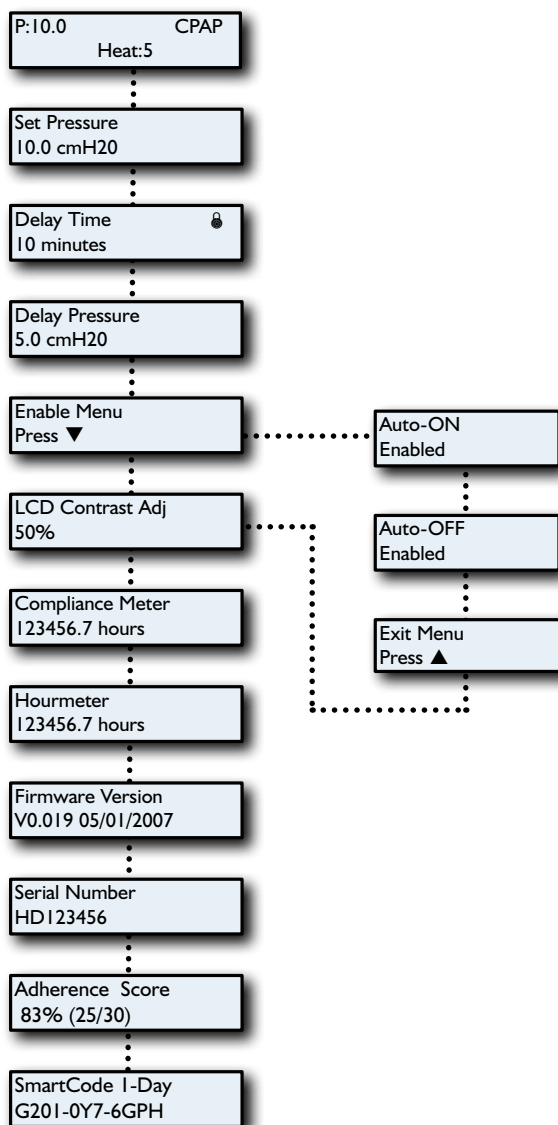## PATIENT MESSAGES

Your SleepCube will alert you of issues that may require some action on your part in order to make your therapy more effective. These messages are displayed when the device is turned on, either by pressing the ON/OFF button or by breathing into the mask. You can press any key or button to clear the message from the display, but the SleepCube will operate normally even if the message is not cleared.

Two kinds of messages are displayed: Reminders and Notifications.

### Reminders

Reminders alert you when parts of your system need to be replaced. Some components of your system wear over time and, if not replaced, may compromise your therapy. If you see a message on the display that is not listed below, you may have the optional DeVilbiss SmartLink™ Module

# TRAVEL/DC OPERATION/SUPPLEMENTAL OXYGEN

attached to your device, which provides additional messages. Refer to the documentation that was provided with the SmartLink Module.

**Clean Filter** – The filter should be checked every 10 days for signs of dirt or wear so that it can be cleaned as needed. This message helps to remind you to check it regularly.

Clean Filter

## Notifications

Notifications identify conditions in your SleepCube that may require action by you or your home care provider in order to maintain a high level of therapy. If you see a message on the display that is not listed below, you may have the optional DeVilbiss SmartLink Module attached to your device, which provides additional messages. Refer to the documentation that was provided with the SmartLink Module.

**NOTE**–The values displayed are examples only.

**Delay Running xx Minutes Left** – While a comfort delay session is active, this message is flashed on the display every 5 seconds to let you know how much time is left in the delay.

Delay Running  
10 Minutes Left

**Mask Off** – This message appears whenever a high airflow condition occurs. High airflow is normally due to a poor mask fit or a removed mask. Check for leak around the mask seal and make adjustments as necessary according to the mask manufacturer's instructions. This message will be displayed until the high airflow problem is corrected. If the high airflow condition persists for about 20 seconds, the device will automatically turn off (if AutoOFF is enabled).

Mask Off. Please  
check mask fit.

**Device Fault** – If a device fault message is displayed, refer to Troubleshooting for instructions.

Device Fault E07  
Call Provider

## SLEEP CUBE TRAVEL INFORMATION

### International Power Changes

Your DeVilbiss SleepCube is equipped with a universal power supply. This means that it is automatically capable of accepting line voltages of 100 to 240V~ 50/60Hz. You do not need to make any adjustments to the device if you are traveling to another country where the line voltage is different than your domestic line voltage. Simply contact your home care provider for the correct style power cord for the region in which you will be traveling (refer to Accessories/Replacement Items for the correct part number).

### Altitude

Your SleepCube automatically compensates for changes in altitude between sea level and 9000 ft (2750 m). You do not need to make adjustments to the device for changes in altitude.

## SLEEP CUBE DC OPERATION

### Battery Power

Your SleepCube can be operated from a 12V DC power source in a motor home, boat or other recreational vehicle. The power source can be used with a 12V DC accessory cable or a 12 V DC stand-alone battery. Both parts are available from your home care provider (refer to Accessories/Replacement Items for the correct part numbers). Both accessory cables are required for stand-alone battery operation. The stand-alone battery connection uses clamps that attach to the terminal connections of the battery.

**NOTE**–The heater for the SleepCube's humidification system will not operate directly from a 12V DC power source. If you need humidification, passover humidification is still available. Alternately, instead of connecting 12V DC power directly to the SleepCube, you can pass the DC power through an inverter that will deliver AC power to the system. This will allow the heater to function. The power rating of the inverter must be at least 200 watts @ 110V~ / 400 watts @ 220V~.

**NOTE**–The SleepCube will automatically switch power sources between AC and 12V DC if both sources are available. If your unit is connected to both AC and 12V DC power sources, it will operate on AC power as long as AC power is present, and automatically switch to 12V DC power if AC power is lost. When AC power is restored, it will return to the AC source. You do not need to reset or adjust any settings when the power source switches, but remember that your heater will not operate on 12V DC.

## SUPPLEMENTAL OXYGEN

Supplemental oxygen is sometimes prescribed in addition to CPAP for treatment of Obstructive Sleep Apnea. If your physician prescribed oxygen for your sleep therapy, you can add the oxygen in one of two ways:

1. Obtain the optional oxygen adapter (part #7353D-601) from your home care provider and connect it to the outlet port on the SleepCube. Connect one end of the air supply tubing to the open end of the adapter and the other end of the air supply tubing to the mask.
2. If your mask is equipped with an oxygen port, your physician may request that you apply oxygen directly to your mask.

**WARNING-OXYGEN USE**

**Oxygen supports combustion. To avoid possible physical injury, do not smoke while using this device with supplemental oxygen. Do not use this device near hot objects, volatile substances or sources of open flames.**

**Always turn on the SleepCube before turning on the oxygen source. Turn the oxygen source off before turning off the SleepCube.**

**This device is not suitable for use in the presence of a flammable anesthetic mixture with air or oxygen, or nitrous oxide.**

**At a fixed flow rate of supplemental oxygen, the inhaled oxygen concentration will vary depending on the pressure setting, patient breathing pattern, mask selection, and leak rate. This warning applies to most types of CPAP devices.**

**Never allow the oxygen source to run continuously while connected to the SleepCube if the device is not in use. If the SleepCube is not being used, turn off the oxygen flow.**

**CAUTION**—Oxygen is a prescription gas and should only be administered under the supervision of a physician.

**CAUTION**—The flow setting of the oxygen source must be specified by a physician.

**ACCESSORIES/REPLACEMENT ITEMS****Replacement Items**

Air-inlet filter (4/pk) .....	DV51D-602
Optional fine particle filter (4/pk) .....	DV51D-603
Air supply tubing (6') .....	7351D-616
Air supply plug .....	DV51D-604
Heater connection cover .....	DV51D-605
Oxygen adapter.....	7353D-601
Carrying case .....	DV51D-610

**Power Items**

AC power cords	
USA.....	DV51D-606
Europe, except U.K. ....	DV51D-607
U.K. ....	DV51D-608
Australia .....	DV51D-609
DC power cord .....	DV51D-619
DC battery clamp-on adapter .....	DV51D-696

**Masks Recommended For Use With The DeVilbiss SleepCube:**

DeVilbiss FlexSet® Mask .....	9354D
DeVilbiss FlexSet® Shallow Mask.....	9354S
DeVilbiss FlexSet® Gel Mask.....	9354G
DeVilbiss FlexSet® Gel Shallow.....	9354GS
DeVilbiss FlexAire® Mask.....	9353D
DeVilbiss FlexAire® Shallow Mask.....	9353S
DeVilbiss Serenity® Mask.....	9352D
DeVilbiss Serenity® Shallow Mask .....	9352S
DeVilbiss Serenity® Gel Mask .....	9352G
DeVilbiss Serenity® Gel Shallow .....	9352GS

## Optional Heated Humidifier

DeVilbiss Heated Humidifier (cradle and chamber) .....DV5HH

## SLEEPCUBE MAINTENANCE

### DANGER

**Electric shock hazard – Do not attempt to open or remove the enclosure; there are no user-serviceable components inside. If service is required, contact your home care provider for instructions on obtaining service of your device. Opening or attempting to service your device will void the warranty.**

### WARNING

**If the operation of the device appears to deviate from normal operation or begins to produce unusual sounds, discontinue use and contact your home care provider for assistance.**

**If the device has been dropped, verify functionality of the device. If the device does not function or the enclosure is damaged, discontinue use and contact your provider.**

**If the device does not appear to be functioning properly refer to the Troubleshooting section for instructions.**

**Pressure Accuracy**–The SleepCube is designed to provide accurate pressure performance from the factory without additional calibration. No routine service is required for a period of 2 years of use, provided the device is used in accordance with the manufacturer’s directions. After 2 years the device should be inspected.

**NOTE**–Some countries require periodic servicing and calibration of this kind of medical device. Contact your home care provider for further information.

**Standard Air-Inlet Filter**–Check the filter every 10 days and clean it if necessary. Replace the filter every 6 months or sooner if it is damaged. The standard filter is designed to filter particles larger than 3.0 microns in size. The SleepCube **MUST** have the standard filter in place during operation.

**Optional Fine Particle Filter**–Check the filter every 10 days, and replace it if it is dirty or damaged; otherwise replace the filter every 30 days. The optional fine particle filter is designed to filter particles as small as 0.3 microns in size and larger.

**Filter Maintenance**–Refer to Cleaning for instructions.

## SLEEPCUBE CLEANING

### WARNING

**To avoid electric shock, always unplug the power cord from the wall outlet power source before cleaning.**

**CAUTION**–Never rinse or place the device in water. Never allow liquids to get into or around any of the ports, switches or air filter; doing so will result in product damage. If this occurs, do not use the device. Contact your home care provider for service.

### Device Cabinet

1. Unplug the SleepCube from the power source. Wipe the enclosure with a clean, damp cloth every few days to keep the enclosure dust free.
2. Allow the device to dry completely before plugging it into a power source.

### Filters

#### Air-Inlet Filter (Standard)

1. The air-inlet filter should be checked every 10 days and cleaned as necessary. The filter is located in the rear of the device.

**CAUTION**–Proper filter function is important for the operation of the device and to protect the device from damage.

2. To clean, remove the dark outer foam filter from the frame.

- Wash the filter in a solution of warm water and dishwashing detergent, and rinse with water. Allow the filter to dry completely. If the filter (part #DV51D-602) is damaged, contact your home care provider for a replacement.
- Ensure the filter is completely dry before installation and use.

### Fine Particle Filter (Optional)

- If you have an optional fine particle filter (part #DV51D-603), inspect this filter and replace it if it shows signs of dirt or damage. Do not wash.
- Install the fine particle filter behind the standard filter.

**NOTE**—If the fine particle filter is not installed first according to the directions, the life of the filter will be reduced requiring more frequent replacement.

### Tubing

The air supply tubing should be cleaned daily. Remove the tubing from the device and mask. Use a mild detergent and water to clean the inside of the tubing. Rinse the tubing and allow to air dry.

### Mask and Headgear

Clean the mask and headgear according to the manufacturer's instructions.

## TROUBLESHOOTING

**Electric Shock Hazard - Do not remove the device cabinet; there are no user-serviceable internal components. The cabinet should only be removed by a qualified DeVilbiss provider.**

If you are having trouble with the operation of your DeVilbiss SleepCube, please refer to the table below for a list of problems with possible causes and remedies. If you are unable to resolve the problem by following the instructions, contact your home care provider for assistance.

Problem	Probable Cause	Remedy
Nothing shows on the display.	<ol style="list-style-type: none"> <li>Device is not plugged in or the power cord is not fully inserted.</li> <li>There is no outlet power.</li> </ol>	<ol style="list-style-type: none"> <li>Verify that power cord is firmly connected to the SleepCube and the power outlet.</li> <li>If you are using a DC power source make sure the cable connections are secure. Verify that your battery is charged.</li> </ol> <ol style="list-style-type: none"> <li>Locate a suitable functioning power source.</li> </ol>
The SleepCube does not start when breathing into the mask.	<ol style="list-style-type: none"> <li>The Auto-ON feature has been disabled.</li> <li>There is no power to the device.</li> <li>Breath is not deep enough for the Auto-ON feature to detect.</li> <li>You are using a full face mask that has an anti-asphyxia valve.</li> <li>The air supply port plug is missing or not fully inserted.</li> <li>The humidifier chamber is not fully engaged into the cradle or is missing.</li> </ol>	<ol style="list-style-type: none"> <li>Use the ON/OFF button to start and stop the device.</li> <li>Verify that the power cord is firmly connected to the device and the power outlet.</li> <li>Breathe deeply in and out to start the SleepCube.</li> <li>Auto-ON may not work because your exhaled breath escapes through the open valve. Use the ON/OFF button to start and stop the device.</li> <li>Make sure the air supply port plug is fully inserted into the unit.</li> <li>Slide the humidifier chamber fully into the slot. Verify the latching lever snaps into place.</li> </ol>
The airflow has stopped unexpectedly during use.	<ol style="list-style-type: none"> <li>The Auto-OFF feature has detected large airflow because of loosely fitting mask.</li> <li>During use your mouth opens and you begin to mouth breathe.</li> </ol>	<ol style="list-style-type: none"> <li>Ensure you have a good mask fit; adjust the mask and headgear as needed.</li> <li>Contact your home care provider for a chin strap or different mask which will help prevent mouth breathing.</li> </ol> <p>If problem is not resolved with above remedies, disable Auto-OFF feature in Advanced Menu options.</p>
The display shows a device fault.	An error has occurred in the device and requires servicing.	Contact your home care provider for service.
Symptoms of OSA have recurred.	<ol style="list-style-type: none"> <li>Air filter may be occluded.</li> <li>Sleep apnea condition has changed.</li> </ol>	<ol style="list-style-type: none"> <li>Clean or replace the air filter and relocate device away from drapes and other dusty surfaces.</li> <li>Contact your physician.</li> </ol>

Problem	Probable Cause	Remedy
Your skin becomes irritated where the mask touches your face.	<ol style="list-style-type: none"> <li>1. Headgear is too tight or improperly adjusted.</li> <li>2. Your mask may not be properly sized or is not the most appropriate shape for you.</li> </ol>	<ol style="list-style-type: none"> <li>1. Loosen the headgear to reduce the contact pressure on your face.</li> <li>2. Contact your physician or home care provider.</li> </ol>
Dryness of throat or nose.	Inadequate humidity.	<ol style="list-style-type: none"> <li>1. Add a DeVilbiss DV5 Heated Humidifier.</li> <li>2. Increase the heater setting on the humidifier.</li> </ol>
Condensation of water is collecting in the hose causing a gurgling noise.	<ol style="list-style-type: none"> <li>1. Your humidification setting is too high.</li> <li>2. The room temperature fluctuates from higher to lower levels through the night.</li> </ol>	Reduce the humidifier heater setting and/or increase the temperature of the room.
Air from flow generator seems to be too warm.	<ol style="list-style-type: none"> <li>1. The air filter(s) is/are dirty.</li> <li>2. The air inlet port is blocked.</li> <li>3. The room temperature is too high.</li> <li>4. The SleepCube is located near a heater.</li> </ol>	<ol style="list-style-type: none"> <li>1. Refer to the Cleaning section for instructions on filter maintenance.</li> <li>2. Unblock the air inlet.</li> <li>3. Reduce the room temperature.</li> <li>4. Locate the device away from the heat source so the air being drawn into the device is at room temperature.</li> </ol> If the problem persists contact your home care provider.
Nasal, sinus or ear pain, possibly with runny nose.	You may have a reaction to air flow pressure.	Discontinue use and contact your physician.

## SPECIFICATIONS

Size .....	10.7 cm x 16.5 cm x 17.5 cm (4.2" H x 6.5" W x 6.9" D)
Weight .....	1.22 kg (2.7 lbs.)
Electrical Requirements.....	100-240V~, 50/60 Hz
Maximum Power Consumption .....	65 watts max from AC power source (flow generator only)
Pressure Range.....	3-20 cmH2O
Operating Temperature Range.....	5°C to 40°C (41°F to 104°F)
Operating Humidity Range.....	0% to 95% RH non-condensing
Operating Atmospheric Conditions.....	Sea level to 9,000 feet (2743 m)
Storage & Transportation Temperature Range.....	-40°C to +70°C (-40°F to 158°F)
Storage & Transportation Humidity Range.....	0% to 95% RH non-condensing
Maximum Limited Pressure .....	20 cmH2O under normal use
Sound Level (tested per ISO 17510).....	26 dBA

### Filter Specifications

- Standard Filter > 3.0 micron particles
- Optional Fine Particle Filter > 0.3 micron particles

### Max Flow Rates

- 83 L/m@ 6.5 cmH2O (1/3 max pressure)
- 134 L/m@ 13 cmH2O (2/3 max pressure)
- 163 L/m@ 20 cmH2O (max pressure)
- Pressure Accuracy .....

### Pressure Swings

(cmH2O peak to peak at 500mL tidal volume, sine wave profile)

Breaths per minute	10	15	20
@ 6.5 cmH2O	0.5	0.6	0.9
@ 13 cmH2O	1.0	1.0	1.2
@ 20 cmH2O	1.0	1.0	1.2

Equipment classification with respect to protection from electric shock..... Class II

Degree of protection from electric shock ..... Type BF Applied Part  
 Degree of protection against ingress of liquids ..... IPX1  
 Equipment not suitable for use in the presence of a flammable anesthetic mixture with air or with oxygen or nitrous oxide.  
 Mode of operation ..... Continuous

**Clinical Specifications for Digital Outputs**

Pressure..... 0 to 25.5 cmH2O ± 0.5

**DECLARATION OF CONFORMITY**

**Manufacturer:** DeVilbiss Healthcare LLC  
**Address:** 100 DeVilbiss Drive  
 Somerset, Pennsylvania 15501-2125 USA

**Product Designation:** CPAP

**Type, Model:** DeVilbiss® IntelliPAP™ or SleepCube™ DV5x Series CPAP  
 Models DV51 Standard CPAP

We herewith declare that the above-mentioned product complies with the requirements of EC Directive 93/42/EEC and the following:

**Class:** IIa, Rule 2

**Quality System Standards Applied:** ISO13485:2003

**Notified Body:** TÜV NORD

**MDD:** Annex II Applied

**Safety Standards Applied:** IEC 60601-1:1988 + A1 1995  
 IEC 60601-1-4:2000  
 ISO 14971:2000  
 EN ISO17510:2002

**EMC Compliance To:** IEC 60601-1-2:2001

**Authorized Representative:** Sunrise Medical Ltd.  
 Sunrise Business Park  
 High Street  
 Wollaston, West Midlands DY8 4PS  
 ENGLAND  
 44-138-444-6688

®Registered U.S. Patent and Trademark Office and other countries.

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©Καταχωρηθέν στην Υπηρεσία Ευρεσιτεχνίας και Εμπορικών Σημάτων Η.Π.Α. και σε άλλες χώρες.
®Registrado en la Oficina de patentes y marcas (Patent and Trademark Office) estadounidense y en otros países.
®Déposé auprès du U.S. Patent and Trademark Office et dans d'autres pays.
®Gedeponoerd bij het U. S. Patent and Trademark Office en in andere landen.
®Registrerad hos U.S. Patent and Trademark Office och i andra länder.
®Registrert ved U.S. Patent and Trademark Office og i andre land.