RESMED

Quattro™ FX

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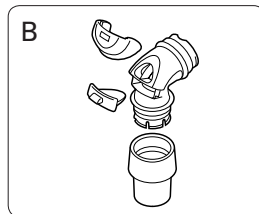
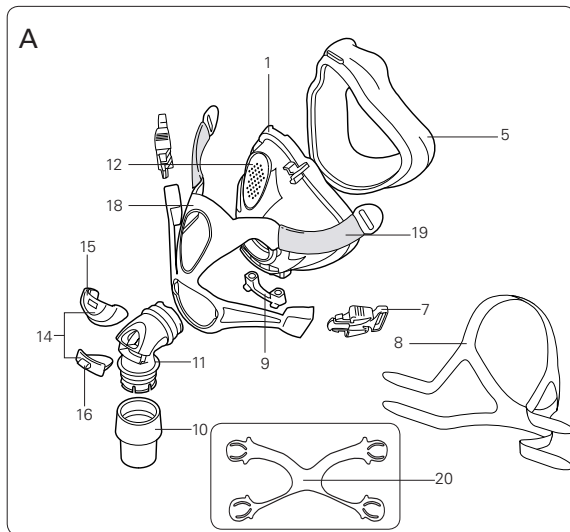
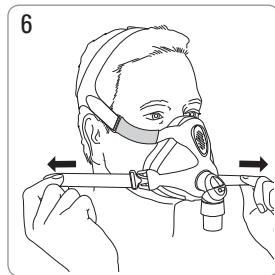
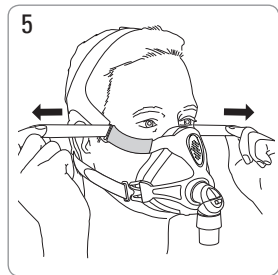
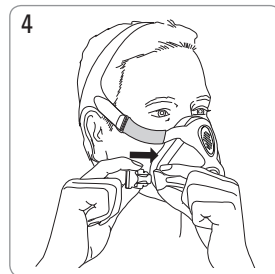
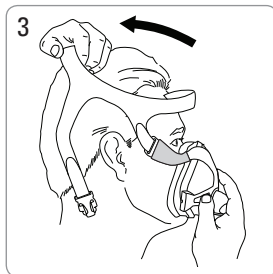
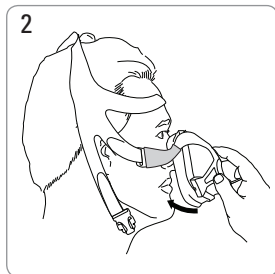
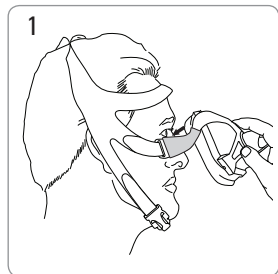
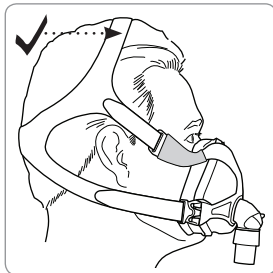
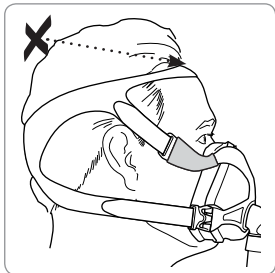
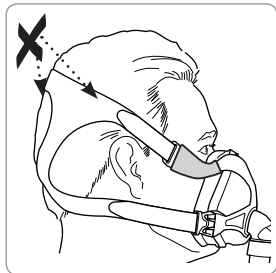
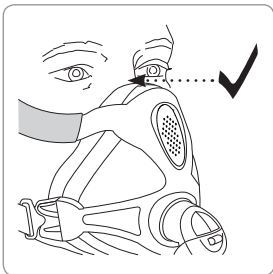
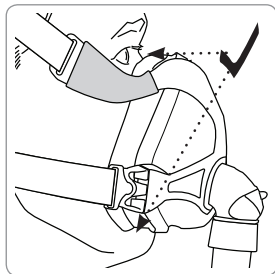
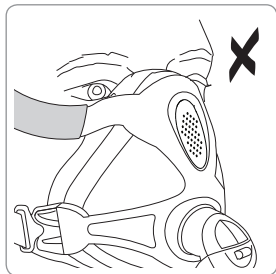
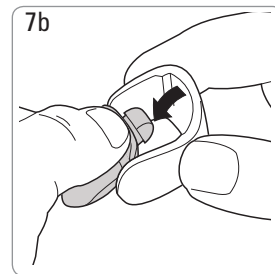
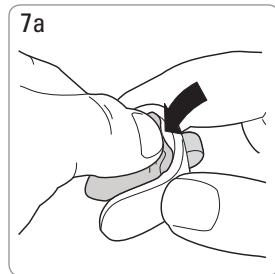
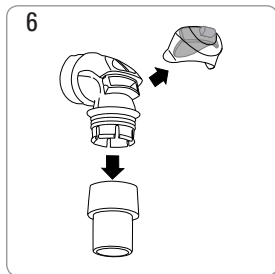
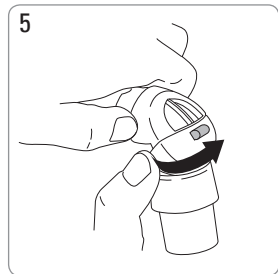
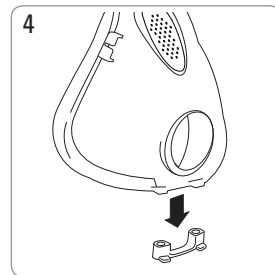
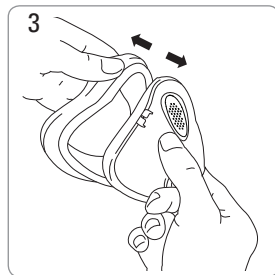
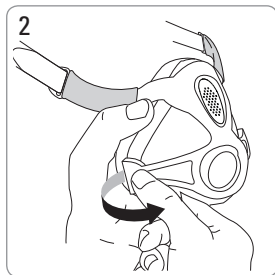
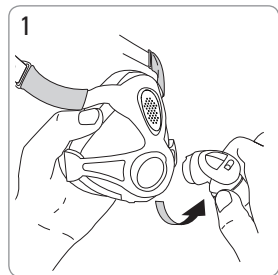
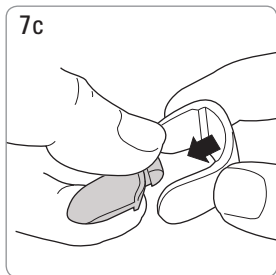
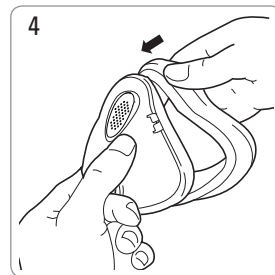
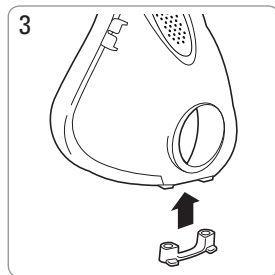
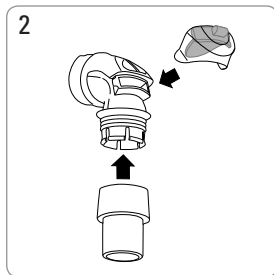
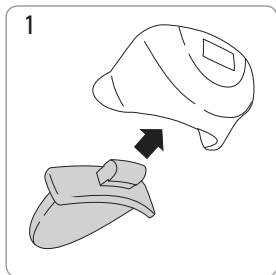
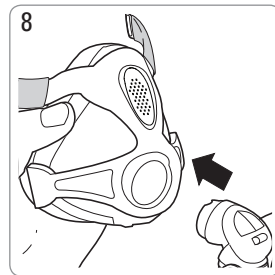
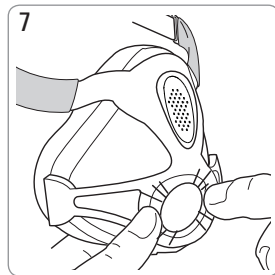
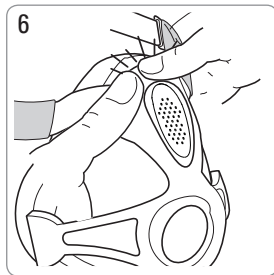
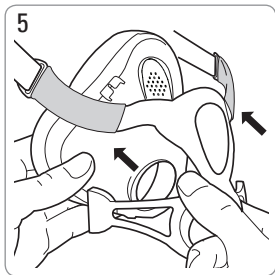
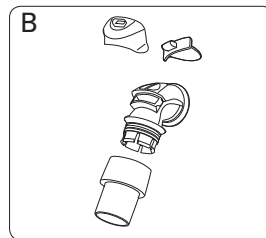
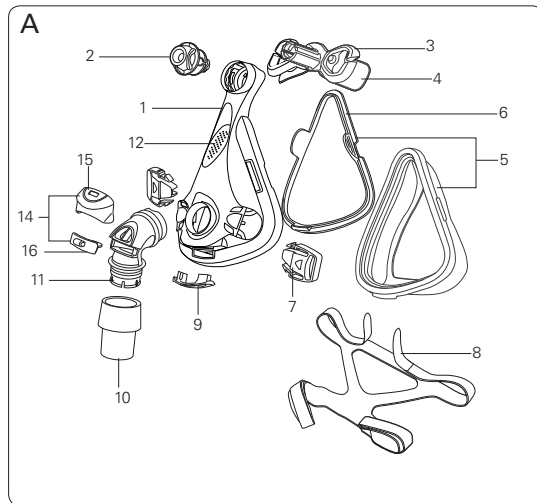
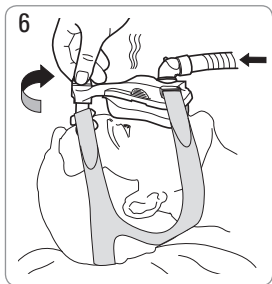
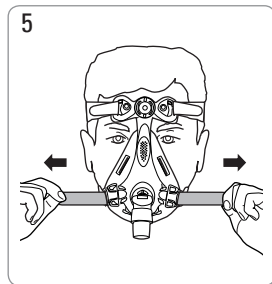
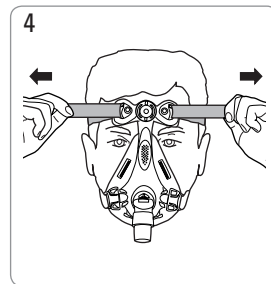
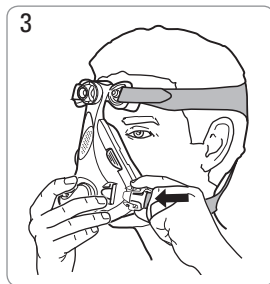
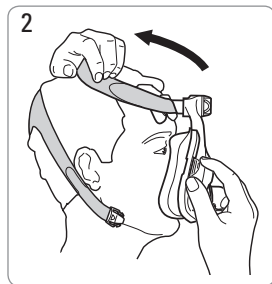
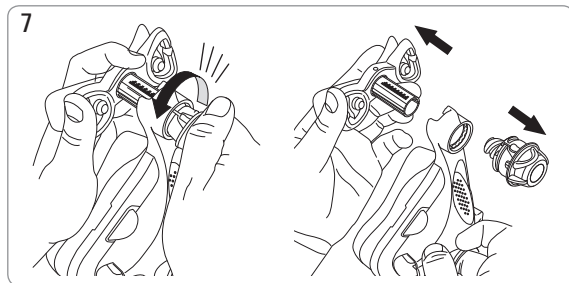
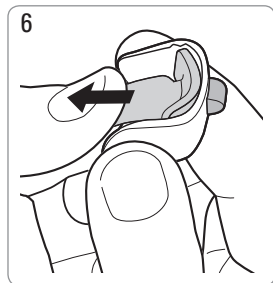
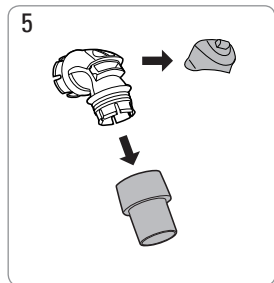
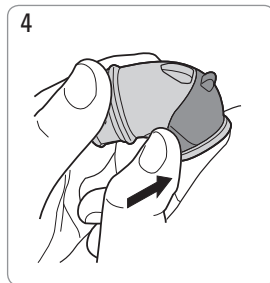
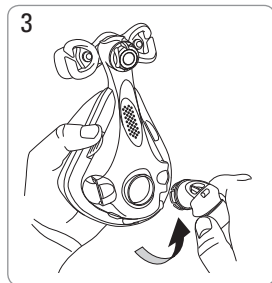
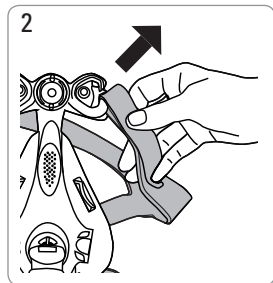
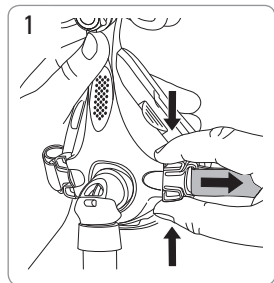
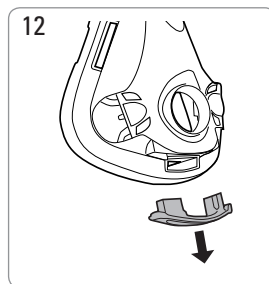
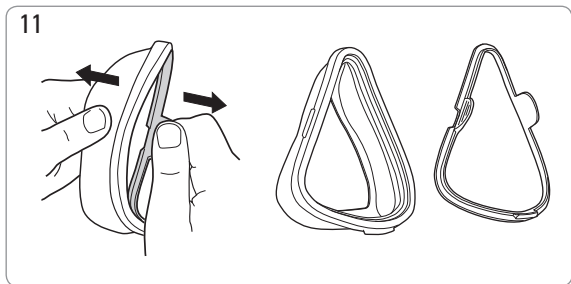
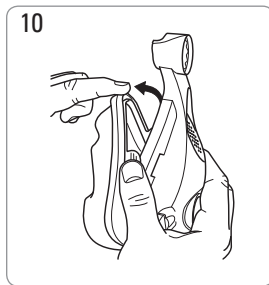
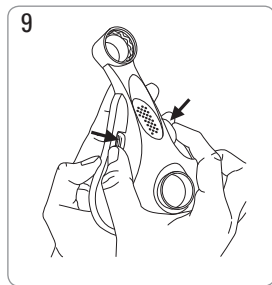
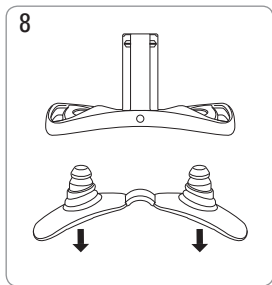
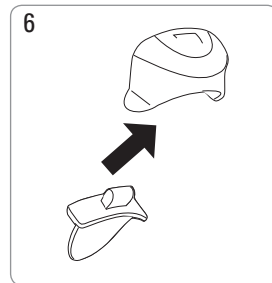
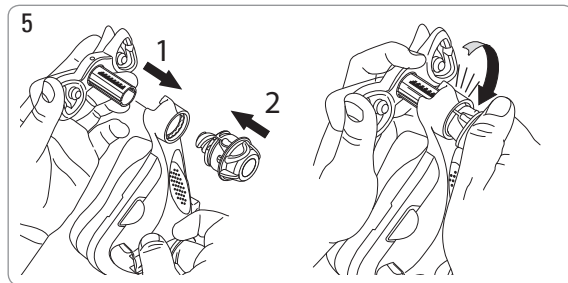
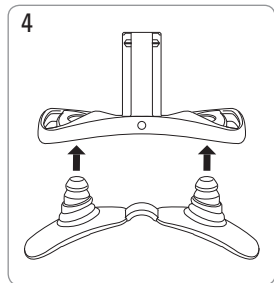
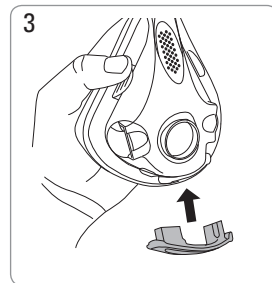
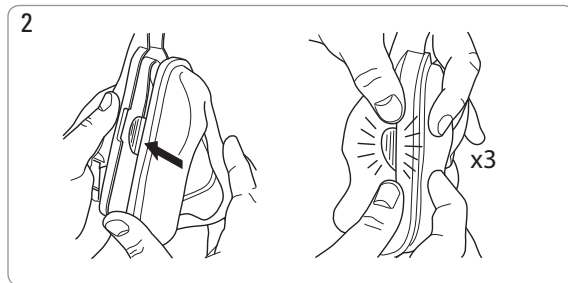
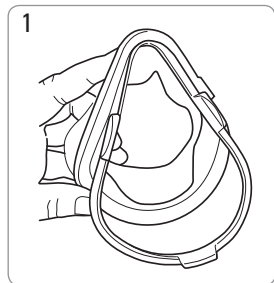
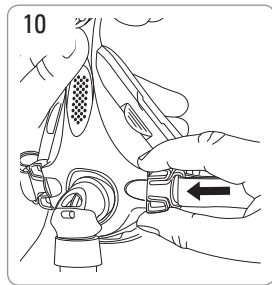
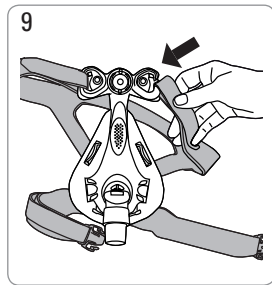
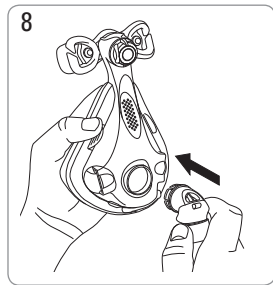
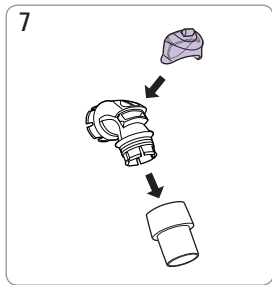
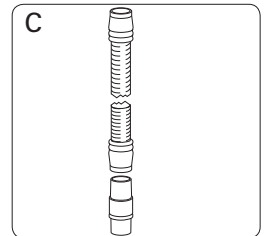
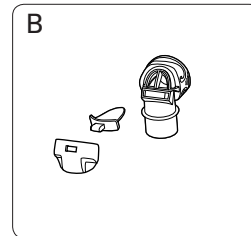
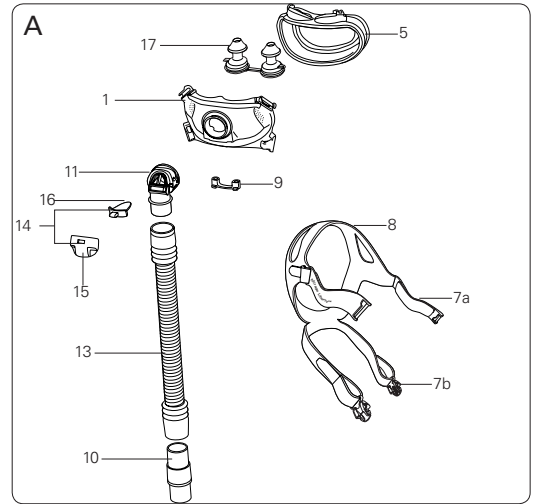
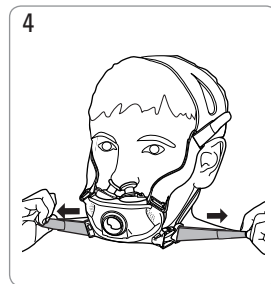
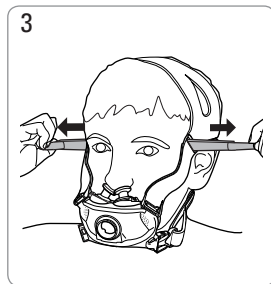
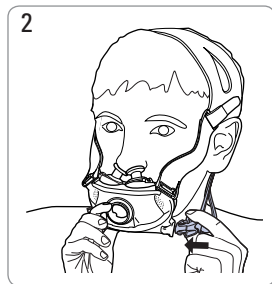
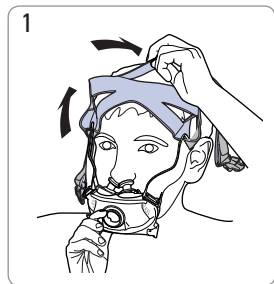
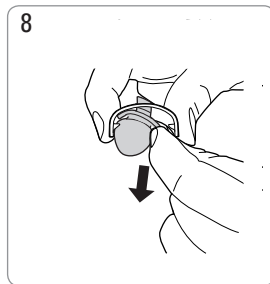
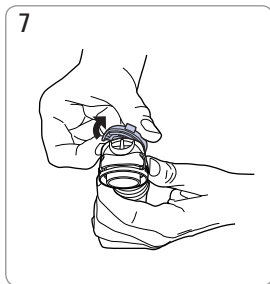
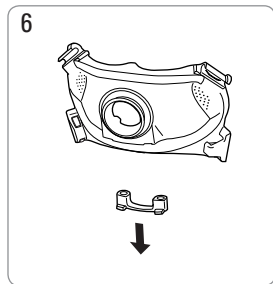
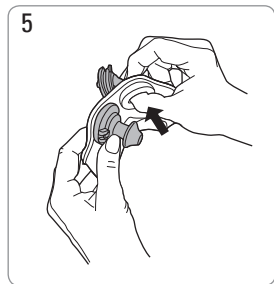
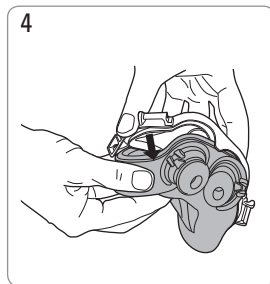
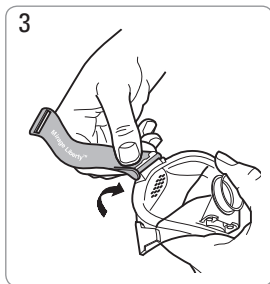
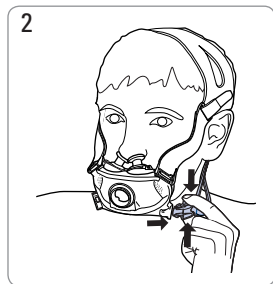
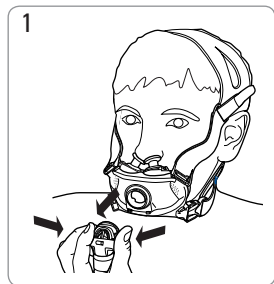
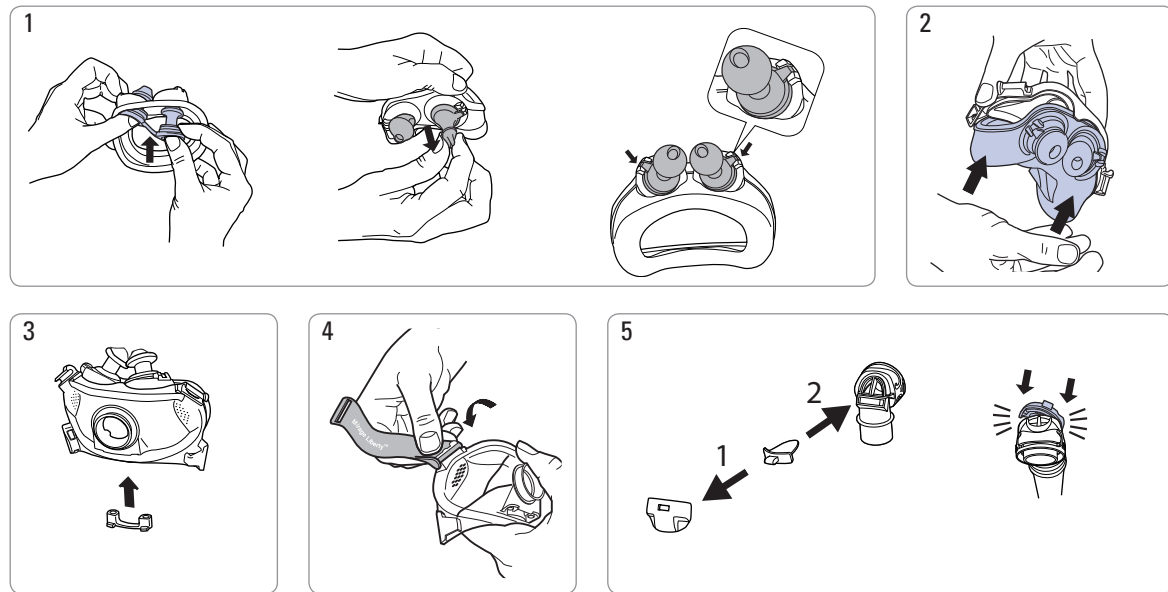
English

Intended Use

The mask directs airflow noninvasively to a user from a positive airway pressure device such as a continuous positive airway (CPAP) or bilevel system.

The mask is intended for:

- adult users (>30 kg) for whom positive airway pressure has been prescribed
- single user re-use in the home environment and multipatient re-use in the hospital/institutional environment.

Using the Mask

- Refer to the Technical specifications section in this user guide for mask setting options.
- Follow the instructions provided by your physician or sleep therapist.
- (Quattro FX/Quattro FX for Her only) Prior to fitting your mask, remove the spring frame support from the rear of your mask and retain for future use.
- To fit your mask, follow the instructions in the Fitting section.
- Do not overtighten the headgear straps. If you feel air-leaks you may need to:
 - refit or reposition your mask
 - adjust your headgear
 - check that your mask is assembled correctly
 - contact your physician or sleep therapist to have your mask refitted.
- Mask-specific user guides are available on www.resmed.com on the **Products** page under **Service & Support**. They are also available from ResMed upon request.

This product does not contain natural rubber latex.

Device Settings

The mask setting options for ResMed devices are listed in the “Mask Compatibility Table” on www.resmed.com on the **Products** page under **Service & Support**.

Disassembly

To disassemble your mask, follow the instructions in the Disassembly section.

Reassembly

To assemble your mask, follow the instructions in the Reassembly section.

Cleaning the Mask at Home

- *The mask and headgear can only be handwashed. The headgear may be washed without removing the headgear clips.*
- *Take care when cleaning to avoid damage or loss of components.*

Daily/After Each Use

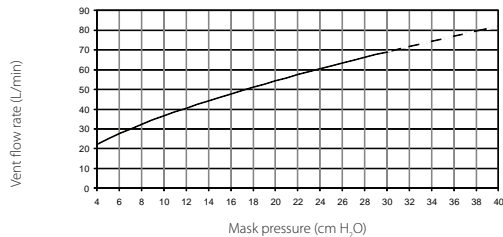
- To optimise mask seal, remove facial oils from the cushion before use.
- Do not soak the mask.
- Handwash the mask by rubbing gently in warm (30°C) water using mild soap or diluted dishwashing detergent.
- Rinse all components well with drinking quality water and allow them to air dry out of direct sunlight.
- If the vent is blocked, clean it using a soft bristle brush.

Weekly

Handwash the headgear in warm (30°C) water using mild soap or diluted dishwashing detergent. Rinse well and allow it to air dry out of direct sunlight.

Technical Specifications

Pressure-flow curve – all masks

Pressure (cm H ₂ O)	Flow (L/Min)		
	Quattro FX/ Quattro FX for Her	Quattro	Liberty
4	22	22	22
8	32	32	32
12	41	41	41
16	48	48	48
20	54	54	54
24	60	60	—
28	66	66	—
30	69	—	—
32	—	72	—
36	—	77	—
40	—	82	—

Deadspace: Empty volume of mask to swivel. The deadspace of the mask varies according to cushion size.

Quattro FX/ Quattro FX for Her	Quattro	Liberty
243mL (Large)	240mL (Large)	151 mL (Large)*

* Empty volume of mask to elbow.

Therapy pressure (cm H ₂ O)			
	Quattro FX/ Quattro FX for Her	Quattro	Liberty
	4 - 30	4 - 40	4 - 20

Resistance: Drop in pressure measured (nominal) (cm H₂O)

	Quattro FX/ Quattro FX for Her	Quattro	Liberty
at 50 L/min	0.1	0.1	0.1
at 100 L/min	0.5	0.4	0.5

Inspiratory and Expiratory Resistance (cm H₂O)

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Sound - DECLARED DUAL-NUMBER NOISE EMISSION VALUES in accordance with ISO 4871. The A-weighted sound power level and the A-weighted sound pressure level of the mask, at a distance of 1M, with uncertainty of 3 dBA, are shown.

	Quattro FX/ Quattro FX for Her	Quattro	Liberty
Power Level	27 dBA	26 dBA	24 dBA
Pressure level at 1m	19 dBA	18 dBA	16 dBA

Gross Dimensions - Mask fully assembled - no headgear. Largest variant only shown.

	Quattro FX/ Quattro FX for Her	Quattro	Liberty
Height	140 mm	206 mm	100 mm
Width	176 mm	109 mm	110 mm
Depth	144 mm	108 mm	100 mm

Mask setting options: Select **A** if available, otherwise select **B**.

	Quattro FX/ Quattro FX for Her	Quattro	Liberty
A	Mir Full	Mir Full	Mir Full
B	Full Face	Full Face	Full Face

Environmental conditions – all masks:

Operating temperature: +5°C to +40°C

Operating humidity: 15% to 95% non-condensing

Storage and transport temperature: –20°C to +60°C

Storage and transport humidity: up to 95% non-condensing

Storage

Ensure that the mask is thoroughly clean and dry before storing it for any length of time. Store the mask in a dry place out of direct sunlight.

(Quattro FX/Quattro FX for Her only) To maintain optimum care of your mask, ensure the spring frame support is fitted when travelling.

Disposal

The mask does not contain any hazardous substances and may be disposed of with your normal household refuse.

The following symbols may appear on the mask or packaging.

 Caution, consult accompanying documents;  Batch Code;  Catalogue Number;  European Authorised Representative;  Temperature limitation;  Humidity limitation;  Does not contain natural rubber latex;  Manufacturer;  Keep away from rain;  This way up;  Fragile, handle with care.

Warnings

- The vent holes must be kept clear.
- The mask should not be used unless the device is turned on. Once the mask is fitted, ensure the device is blowing air.
- Follow all precautions when using supplemental oxygen.
- Oxygen flow must be turned off when the CPAP or bilevel device is not operating so that unused oxygen does not accumulate within the device enclosure and create a risk of fire.
- At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration varies depending on the pressure settings, user breathing pattern, mask, point of application and leak rate.

- If any visible deterioration of a mask component is apparent (cracking, tears, etc), the mask component should be discarded and replaced.
- Do not use aromatic-based solutions or scented oils (eg, eucalyptus or essential oils), bleach, alcohol or products that smell strongly (eg, citrus) to clean any of the mask components. Residual vapours from these solutions can be inhaled and may also damage the mask, causing cracks.
- Your physician or sleep therapist is responsible for ensuring that the mask is compatible with the device. If used outside specification or if used with incompatible devices, the seal and comfort of the mask may not be effective and optimum therapy may not be achieved.
- Do not combine components from different mask systems.
- Discontinue using the mask and consult your physician or sleep therapist if you have ANY adverse reaction.
- Refer to your CPAP or bilevel device manual for details on its settings and operational information.
- This mask should be disinfected and/or sterilised when used between users.
- Remove all packaging before using the mask.
- The elbow and valve assembly have specific safety functions. They allow you to breathe fresh air if the airflow from your CPAP or bilevel device to your mask is impeded. The mask should not be worn if the valve is damaged as it will not be able to perform its safety function. The valve should be replaced if it is damaged, distorted or torn.
- The mask must be used under qualified supervision for users who are unable to remove the mask by themselves. The mask may not be suitable for those predisposed to aspiration.
- (Quattro FX/Quattro FX for Her only) Ensure the soft sleeves are fitted at all times during use.
- Using a mask may cause tooth, gum or jaw soreness or aggravate an existing dental condition. If symptoms occur, consult your physician or dentist.

Consumer Warranty

ResMed acknowledges all consumer rights granted under the EU Directive 1999/44/EG and the respective national laws within the EU for products sold within the European Union.

Clinical Information

The mask contains passive venting to protect against rebreathing.

Technical specifications, cleaning, disinfection and sterilisation instructions are available in the mask specific user guide on www.resmed.com on the **Products** page under **Service & Support**

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©2012 ResMed Ltd.
Mirage, Mirage Quattro, Quattro and Mirage Liberty are trademarks of ResMed Ltd and Mirage is registered in U.S. Patent and Trademark Office. Velcro is a trademark of Velcro Industries B.V.
ResMed Ltd 1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia
ResMed Corp 9001 Spectrum Center Boulevard San Diego CA 92123 USA
ResMed (UK) Ltd 96 Milton Park Abingdon Oxfordshire OX14 4RY UK